

The C.A.S.A. Vision Eye Chart

Screener (DISTANCE TEST)

Test vision differences in one eye at a time and with both eyes open.

The C.A.S.A. vision eye chart is a tri-colored distance chart designed to detect changes in vision. The background colors employed on this chart are green, red and yellow. These are the diagnostic colors involved in chromatic aberration of the eye.

SEEING CLEARER IN THE RED MAY INDICATE

A MYOPIC SHIFT WITH OR WITHOUT

ASTIGMATISM. SEEING CLEARER IN THE GREEN

MAY INDICATE A HYPEROPIC SHIFT WITH OR

WITHOUT ASTIGMATISM. SEEING CLEARER IN

THE YELLOW MAY INDICATE A MIXED

ASTIGMATIC SHIFT.

N.B. Seeing clearly in every color does not mean that the vision is normal. Certain visual anomalies may still exist. This visual screening technique does not replace a full eye exam. C.A.S.A. Vision recommends yearly eye examinations at the eye doctor.

© Copyright C.A.S.A. Vision Inc 2012

C.A.S.A. Vision Inc., its members and affiliates assume no liability for any advice, diagnosis or other information or services that you obtain on this pamphlet (including our websites)

www.casa-vision.com • www.casavisioneyechart.com

BLURRY VISION MAY ALSO

BE ASSOCIATED WITH

OTHER OCULAR DISEASES.

How to use the C.A.S.A. Vision

Distance Chart (Screener)

(Repeat for each eye)

Note: If you wear distance glasses (or contact lenses), be sure to have them on when using the C.A.S.A. Vision test. Place yourself 10 feet (3 meters) from the distance eye chart.

- Look at the letters (words) in each color with one eye at a time covering the other eye with your hand.
- Make sure you stay focused on the clarity of the letters (words) in each color.
- If the letters (words) are blurry in one color or more, your vision may not be safe for driving. Report to your eye doctor.

**IF THE LETTERS (WORDS)
ARE CLEARER IN THE GREEN COLOR,
(OR BLURRY IN THE RED COLOR)**

**CONTACT YOUR EYE DOCTOR
IMMEDIATELY**

**IF YOU HAVE ALREADY BEEN
DIAGNOSED WITH DRY MACULAR
DEGENERATION (AMD) IN THE PAST,**

2729 Villeray, Montréal, Québec, Canada H2A 1E4

For more info: www.casavisioneyechart.com

C.A.S.A.

Vision Inc

C.A.S.A. Vision

Diagnostic Screener

== (SELF TESTER) ==

NOTE:

Use only the most recent prescription if you wear distance glasses or contact lenses.

RECOMMENDED

VISION MONITORING

FOR AGE-RELATED

**MACULAR DEGENERATION
(AMD)**

THIS VISION SCREENING

PROCEDURE CAN BE DONE

ON A DAILY BASIS

ESTABLISHMENT LICENCE 4310

HEALTH CANADA MEDICAL DEVICE CLASS 1

FDA