

# THE CASA VISION EYE CHART SCREENER (NEAR TEST)

The Casa Vision eye chart is a tri-colored eye chart designed to detect changes in vision. This self-testing vision chart which is comprised of Snellen letters varying from 20/200 to 20/20 is used for near vision.

The background colors employed on this chart are green, red and yellow.

These are the diagnostic colors involved in chromatic aberration of the eye.

The relative clarity in one color with respect to the others relays information as to the visual state of the observer, hence, lending diagnostic value to the test.

*Use your near prescription when using the vision chart.  
Testing distance is done at 16" (40 cm) from the observer's eyes.  
Test vision differences in one eye at a time and with both eyes open.*



*If the observer sees certain letters blurry in 1 color or more, please go see your eye doctor.  
The Casa Vision eye chart screener is not a substitute for regular eye exams at the eye doctor.*

***IF THE LETTERS ARE CLEARER IN THE GREEN COLOR,***

***(OR BLURRY IN THE RED COLOR)***

***CONTACT YOUR EYE DOCTOR IMMEDIATELY IF YOU HAVE ALREADY BEEN DIAGNOSED  
WITH DRY MACULAR DEGENERATION (AMD) IN THE PAST.***

**RECOMMENDED VISION MONITORING FOR**

**AGE-RELATED MACULAR DEGENERATION (AMD).**

**THIS VISION SCREENING PROCEDURE CAN BE DONE ON A DAILY BASIS.**

---

*BLURRY VISION MAY ALSO BE ASSOCIATED WITH OTHER OCULAR DISEASES.*

*C.A.S.A. Vision Inc., its members and affiliates assume  
no liability for any advice, diagnosis or other information  
or services that you obtain on our websites.*

ESTABLISHMENT LICENCE 4310  
HEALTH CANADA MEDICAL DEVICE CLASS 1  
FDA